

## Alert-iT Alarm Message Responses



Name of user **Alarm:**

**URGENT** = Life critical alarm: Attend immediately

**HELP** = Comfort critical alarm: Attend with speed

**ASSIST** = Non critical alarm

**FAULT** = Equipment needs servicing


### Special Fault Alarms

**RF FAIL:** Correct immediately as radio contact has been lost with the user's equipment

**FAULT30:** The user's battery needs charging or replacing, when convenient

**FAULT31:** The user's battery is almost exhausted and needs charging re replacing immediately

### Action:

1. The pager sound can be muted by pressing 
2. Attend the user.
3. Reset the alarm on the user's equipment (see examples below)



Press to reset



Press for 1 second until LED stops flashing (and beep heard on some units)


Press *Disarm* to cancel



Full adjustment details are found in handbooks available on:  
[www.alert-it.co.uk/handbooks/](http://www.alert-it.co.uk/handbooks/)  
 Or by phoning Alert-iT

# Basic User Pager Information


Typical screens in normal operation

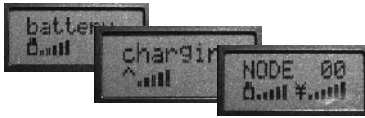

OR



Press 3 sec to enter setup

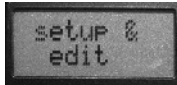

Press to silence alarms



Scroll to menu item required eg.



(shows a list of all alarm units)



*Turn off may require a password depending on the security level set by the Supervisor*




Select item

etc


Backlight



Vibrate Mode

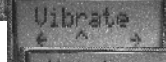


Beep on key press

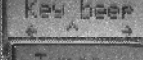

Sound on Alarm



Sound if RF Fail

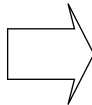

Select any item to change



Use keys to select on or off



Press select to save any change or Menu to abort



The RF Fail warning could be a normal daytime occurrence if the bedside units are turned off. Disabling the tunes stops the annoyance of the audible warning, but the display will still warn on the situation