


## Quick Start Guide to using the P137/P138 Care Pager

Press for 3 sec to turn on.  
If no response then charge battery  
for at least 5 minutes

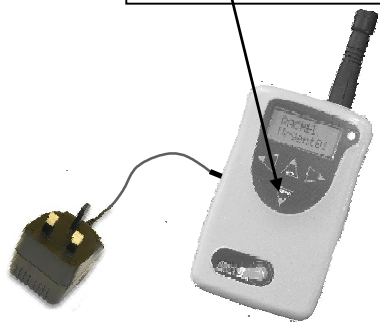

Charger can be left permanently connected or it can boost charge from empty in 8 hours

### Typical screens in normal operation



Shows battery state on bars

Name of alarm will flash up when safety signal received. Bars show strength of signal




Indicates there is an alarm system in the area, not coupled to this pager. This may be OK or an error so check with the supervisor

### Alarm active



Radio contact with an alarm fitted with "Safelink" security has been lost. This MUST be corrected immediately

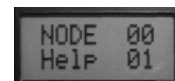

Typical alarm indication, with four levels: Urgent, Help, Assist, Fault

Try walking towards the user's room. If the alarm cancels then there is a problem with the building or distance required. Contact iTs Designs who can offer a signal booster. If the alarm does not cancel then:

1. Check the user equipment is turned on.
2. Check the user equipment is transmitting a regular safety signal (typically the green power light will flash every 8-10 seconds)
3. Contact iTs Designs for further advice

1. Attend the user,
2. Reset the alarm on the user's equipment
3. and the pager will return to normal operation automatically

*PTO for setup options*

